

210\*148mm



# ORAL FEEDING SYRINGE

English

MHK024

**Product Name:** Oral Feeding Syringe  
**Product Materials:** Food Grade Silicone, Non-toxic PP  
**Product Dimensions:** 12.7cm x 4.7cm  
**Product Volume:** Syringe: 10ml  
Measuring Cup: 20ml

**Age Guide:** 0 months+

## Product Description:

The Haakaa Oral Syringe is the perfect way to administer milk, liquid diets and larger quantities of medicine to premature or just plain fussy babies in a mess and waste-free way. The dual-angled ports on either side of the nipple shoot liquid into the sides of your little one's mouth instead of the back of their throat to greatly reduce the risk of choking. To use the syringe, measure the proper dose of medicine by pulling the plunger while the tip is submerged in liquid, then double-check the amount. Position your child in your lap facing away from you, then place the syringe in their mouth. After your baby starts to suck, slowly push the syringe plunger in.

## Features:

- Made of food grade silicone and non-toxic PP plastic.
- Dual angled ports in the nipple tip disperse medication sideways into your child's cheeks instead of the back of the throat.
- Soft, pacifier-style teat is easy to feed from.
- Pacifier-style shield prevents potential choking hazards.
- Clearly marked measuring bar ensures accurate measurements.
- BPA, PVC and phthalate-free.


## Cleaning and Care:

Do not use any bleach-based cleaners or tablets to sterilise or clean your Haakaa products. To sterilise, use a steam steriliser or boil in water for 2-3 minutes. PP parts should not be boiled.


## Caution:

- Check product condition regularly. Replace if this product shows any signs of damage.
- Do not store near any sharp objects.
- Only soft bristle brushes or soft sponges should be used to clean this product. Hard scourers may scratch the surface.
- Store product in a cool, dry place and avoid direct sunlight.
- Adult supervision is required.


## Instructions:

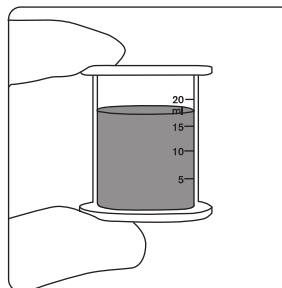


- 1 Measure the proper dose of liquid in the syringe cap, then double-check the amount.

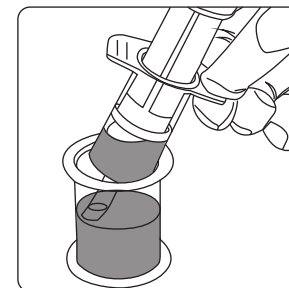

- 2 Use the syringe to draw the liquid out from the cap by submerging the tip in the liquid and pulling the plunger back.

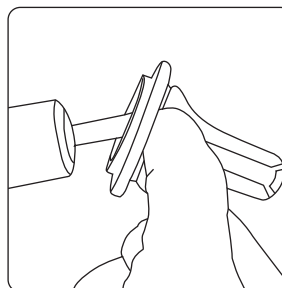

- 3 Once you have drawn the required amount of liquid, place the syringe tip inside the opening of the silicone nipple and push in firmly to attach.

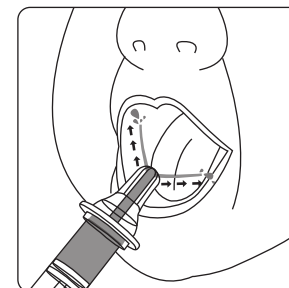

- 4 Introduce the syringe into your child's mouth, and after they start to suck, slowly push the syringe plunger in.